

GUTTAPLAST

Liquid for root canal desobturation

Composition

Eucalyptus oil
Citral

Indications and properties

Repeated endodontic treatment of root canals previously sealed with gutta-percha.

In cases of repeated endodontic treatment a special liquid for unsealing the root canals sealed with gutta-percha is used, which facilitates such manipulations.

Recommended use

Method for unsealing the root canals in one visit.

Proceed with the mechanical treatment of pulp chamber and the entrance to the root canals as widely as possible. Using a probe introduce two drops of "Guttaplast" in a tooth cavity, and then into the root canals and leave it for 1 minute in order to soften a gutta-percha.

Then start removing softened gutta-percha from the root canal with endodontic instrument as long as you reach the apex. From time to time add fresh solvent and wash the root canal with sodium hypochlorite.

Method for unsealing the root canals in two visits.

Proceed with the mechanical treatment of pulp chamber and the entrance to the root canals as widely as possible. Wet a cotton tampon with the liquid and place it to the root canal, slightly compacted. Close the root canal with temporary filling until the next visit. After 1-3 days remove the temporary filling and proceed with mechanical treatment to remove softened gutta-percha. It would not be difficult. After unsealing the root canal should be washed, degreased, and dried in order to prepare for subsequent filling.

Attention! Avoid falling into the esophagus.

Package contents and storage

Liquid is available in a glass bottle of 13 ml.

Attention! The liquid is flammable. Keep away from the heat or flame sources in room temperature away from direct sunlight.

Shelf life is 3 years.