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
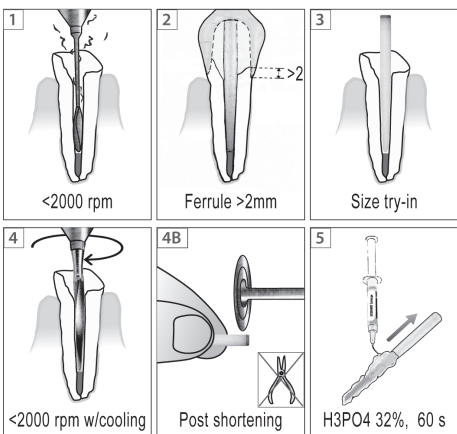
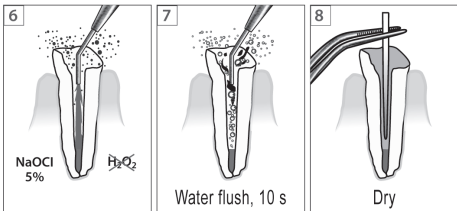
Overfibers Post clinical Use


Canal Disinfection

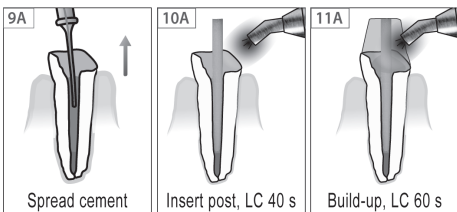A. PROCEDURE FOR SELF-ADHESIVE CEMENTS

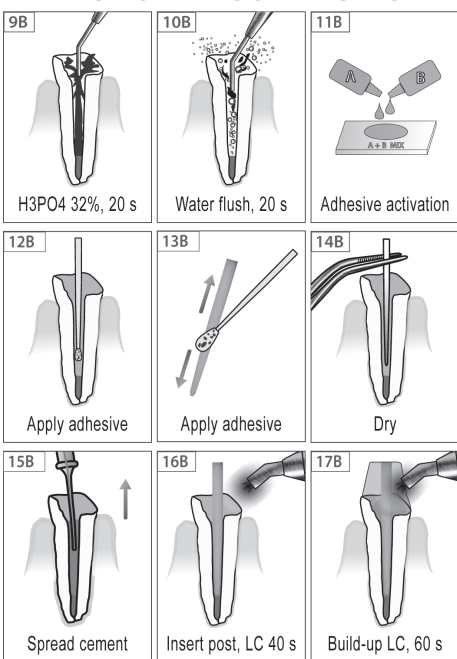B. PROCEDURE FOR RESIN CEMENTS & ADHESIVES

Easy Removal, Translucent and Radiopaque type "S" glass fiber endodontic post.

They are composed of type "S" high-strength glass fibers in an epoxy resin matrix. They are preferably inserted after preparation of the canal with Prosthetic Drills.

If, for any reason, it is necessary to remove a Hi-Rem Prosthetic Post from the root canal, Hi-Rem Prosthetic Post has a soft, colored macro-fiber that allows guided and safe post removal through an internal reaming procedure (See Removal of Hi-Rem Prosthetic Post)

Sizes and color codes

Hi-Rem Prosthetic Posts (19 mm in length) and Prosthetic Drills, are available in four sizes, with color codes for a corresponding use:

Hi-Rem Prosthetic Post				Prosthetic Drill
Color	Max Ø	Min Ø	Taper (%)	Color
Black # 0	1,20	0,70	0,04	Black # 0
Yellow # 1	1,40	0,80	0,05	Yellow # 1
Red # 2	1,60	0,80	0,066	Red # 2
Blue # 3	1,80	0,90	0,08	Blue # 3

Tab. I

Indications

Hi-Rem Prosthetic Posts, conical-cylindrical shape, have been designed to suit all clinical cases and in particular teeth that are extensively destroyed and very stressed. The available sizes allow all dental typologies to be treated (Tab. II).

Post Code	Teeth
Black # 0	Inferior incisors
Yellow # 1	Inferior and superior incisors, first superior premolars
Red # 2	Superior incisors, canines, premolars, molars
Blue # 3	Superior incisors, canines, premolars, molars

Tab. II: General indications

Steps for clinical use

1. Remove all the sealing material until the calculated depth using endodontic drills such as Gates-Glidden® or Largo® at low speed (1500-2000 RPM). Minimize removal of canal dentin. At least 3 - 4mm of the apical seal must remain to guarantee endodontic success.

2. For prosthetic restorations: at least 2 mm of dentin must remain above the foreseen preparation margin to guarantee the "ferule" effect;

3. Select the post to be used based on the type of tooth and the width of the canal (Tab. II); the post must be the largest still compatible with the canal diameter; the portion that enters into the canal must be at least half of the post total length.

4. Once the post has been chosen, prepare the apical portion of the canal with Prosthetic Drill, refer to table I.

Prosthetic Drill must be sterilized prior to use.

- Shorten the coronal portion of the post, bringing it to the desired length. Use only diamond rotating instruments of medium or large granulometry.

Carry out this operation far from the patient, using gloves, a mask, and protective goggles.

5. Clean the post by immersing it in pure ethyl alcohol or coating it with 32% phosphoric acid gel for 60 seconds; then wash it with plenty of water for at least 30 seconds and dry it until it became matt white.

6. As Overfibers post surfaces are particularly rough, silanization treatments are optional.

7. Hi-Rem Posts can be used with auto-polymerizing or dual-curing adhesive/cement systems. Choose the preferred adhesive/cement system and carefully follow the manufacturer's recommendations.

8. Reconstruction of the stump: choose the desired reconstructive system and carefully follow the manufacturer's recommendations. Place the composite directly on the external part of the post, previously

Hi-Rem Prosthetic Post

treated with the adhesive system. Do not create empty spaces. CAUTION: temporary cements or disinfectant substances containing phenols (e.g. eugenol) previously applied to a tooth can greatly reduce the bond to the dentin and the strength of resin cements and composites!

9. After tooth preparation, apply the PROVISIONAL RESTORATION, avoiding cements or disinfectants that contain phenols (e.g. ZOE cements).

Removal of Hi-Rem Prosthetic Post

1. Expose the post with a diamond bur to the coronal level until the colored macro-fiber will be clearly identified at the center of the post.

2. Use a new, thin (0.25 - 0.3mm) Ni-Ti rotating file at medium speed and drill gently into the macro-fiber with up and down movements.

3. Allow the instrument to penetrate for 1.5 - 2mm and retract it after 3 - 4 seconds to allow the removal of the melted shavings. Avoid excessive friction heat build-up, but do not use cooling spray.

4. Repeat the procedure until the depth established on the radiograph is reached. Since the macro-fiber is very soft, the instrument is guided towards the post tip by the harder walls of the post, without ever deviating from the central axis. During this phase, the operator can clearly feel that the instrument is conducted through the post by the macro-fiber, thereby assuring a sense of certainty.

5. After drilling through the macro-fiber, a canalized post is obtained: grind out the internal walls of the post with new, typical Gates-Glidden®, Largo®, or Peseo® reamers, until cement or dentin is reached. Beware the excessive friction heat!

6. Remove the remaining cement and other debris using proper ultrasonic tips and make a radiograph control to be sure the old cement is utterly gone. Now the removal of the post can be said to be finished: rapidly, safely, without sacrificing healthy dental tissue.

7. Proceed with the conventional endodontic treatment followed by the proper restorative practices.

Precautions

To obtain the safety data sheets (Overfibers MSDS), go to www.overfibers.com website.

Storage

The product must be kept in a cool, dry place protected from sunlight. The device is disposable.

Disposal

Dispose in accordance with the regulations in force.

Warnings and contraindications

Do not use in patients with known or suspected allergies against the materials constituting the device.

- Hi-Rem Prosthetic Posts can withstand a sterilization procedure in envelopes through autoclaving at 135°C and 2.1 bar of pressure for 5 minutes. This procedure must be applied in all cases of suspected or established biological contamination of the post.

- Avoid the prolonged and repeated breathing of dust arising from the abrasion of the device during clinical procedures. The use of protective masks, high-velocity suction, and rubber dams is recommended.

- Keep away from children.

- The product is intended solely for dental use.

Manufacturer: Overfibers S.r.l - Via Malatesta, 7 - 40026 - Imola (Bo) Italy

1) This material has been developed solely for use in dentistry. It must be employed by exclusively following the instructions for use. The manufacturer assumes no liability for damages resulting from the failure to observe the instructions for use or for a use other than the scope foreseen for this product. Users are responsible for testing the material in usage fields not explicitly mentioned in the instructions for use. The descriptions and data do not constitute any guarantee of the product's properties and are not binding.

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Producent: Overfibers S.r.l - Via Malatesta, 7 - 40026 - Imola (Bo) Italy