

POLISHPASTE-D-DRY GLOSS
Paste with diamond filler for final polishing of composites and ceramics to dry gloss

Indications

Polishing paste for final polishing to dry gloss of fillings made of composite materials light- and self-cured and ceramics.

Composition

Diamond abrasive
Water soluble polymer base
Silicone additives
Stabilizer
Flavoring agent

Properties

The action of "Polishpaste-D-Dry gloss" based on the specific properties of the fine abrasive diamond powder.

Because of these properties diamond abrasive produces final polishing of fillings made of composite materials.

Recommended use

Before starting the procedures the surface should be slightly dried. Apply to the polishing cup or brush the required amount of "Polishpaste-D-Dry gloss". The procedure should last 1-3 min at medium speed, not exceeding 8-10 thousands rpm and without excessive pressure on the surface. Polish until the filling has dry gloss. After the end of the polishing wash out residual paste with water.

Package contents and storage

"Polishpaste-D-Dry gloss" is available in 2 syringes by 3 ml.
Recommended temperature of storage is +4°C to +24°C.
Shelf life is 3 years.